

Advazorb® Heel

Hydrophilic foam dressing with film backing

Fact sheet


Product description

Advazorb® Heel is a conformable, non-adherent, hydrophilic, polyurethane foam dressing with a breathable film backing designed to fit around the heel.

A non-adherent foam dressing with excellent fluid handling properties. Ideal for use under compression bandaging with its fluid retention, low profile and low friction film backing.

Use

Apply directly to the wound on the heel, pink side up and secure in place with tape, appropriate bandage or secondary dressing.

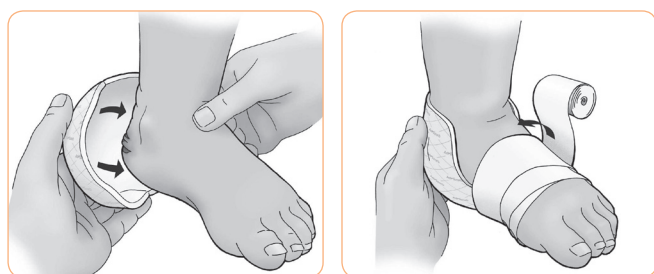

Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Advazorb® Heel ordering information

Size	Stock code	Pack size	NHS code	PIP code
17cm x 21cm	CR4228	5	ELA625	374-4646



Features









- ✓ Sterile Hydrophilic PU foam
- ✓ Low friction thin film backing
- ✓ Bacteria proof backing
- ✓ Soft and conformable
- ✓ High fluid retention
- ✓ Anatomically shaped

Indications

Suitable for acute and chronic exuding wounds including:

- ✓ Cuts and abrasions
- ✓ Superficial burns
- ✓ Surgical wounds
- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Diabetic ulcers

Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize



TOP TIP

Advazorb® Heel can be turned inside out and used for wounds underarm.

Advancis
Medical
www.advancis.co.uk