


## Super absorbent dressing with a soft silicone contact layer

### Product description

Eclipse® Adherent is a super absorbent wound exudate management dressing with a soft silicone contact layer. The soft silicone layer provides convenient gentle adherence and will reduce pain and tissue damage on removal.

### Use

Eclipse® Adherent is placed face down on wound surface with beige backing uppermost. For large wounds several dressings can be placed side-by-side. Wear time will depend on the level of exudate but Eclipse® Adherent can be left in place for up to seven days.

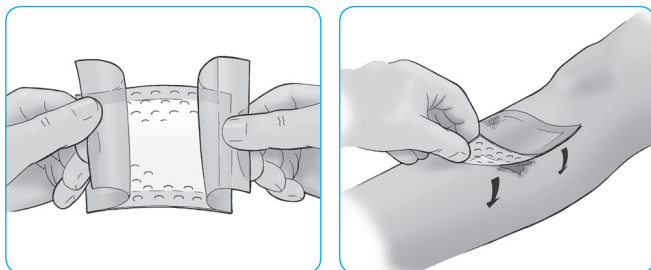

### Contra-indications

Arterial bleeds and heavily bleeding wounds.  
Do not cut Eclipse® Adherent.

### Storage

Store at room temperature, out of direct sunlight and keep dry.

### Ordering information for Eclipse® Adherent

Size	Stock code	Pack size	NHS code	PIP code
10cm x 10cm	CR3881	10	EKH034	325-2293
10cm x 20cm	CR3883	10	EKH011	325-2269
15cm x 15cm	CR3863	10	EKH036	325-2277
20cm x 30cm	CR3864	10	EKH015	325-2285

### **i** TOP TIP

A good tip is to choose a larger size of Eclipse® Adherent than you would usually select. Due to the hydrophobic nature of soft silicone, the dressing will need to adhere to a dry surface that surrounds the wound.



### Features

- ✔ High capacity exudate management
- ✔ Atraumatic dressing technology
- ✔ Breathable fluid repellent backing
- ✔ Moist wound healing environment
- ✔ Moisture locking system

### Indications

Moderate to heavily exuding wounds including:

- ✔ Leg ulcers
- ✔ Pressure ulcers
- ✔ Sloughy or granulating wounds
- ✔ Post operative sutured or dehisced wounds
- ✔ Fungating wounds
- ✔ Donor sites

### Care and use symbols

	Read instructions for use		CE marked medical device
	Single use only		Keep dry
	Do not use if packaging is broken or damaged		Store out of direct sunlight
	Sterilised using ethylene oxide		Do not re-sterilize