


Product description

Eclipse® is a highly absorbent, high capacity wound exudate management product designed to absorb fluid, reduce potential leaks and reduce risk of maceration.

The dressing has a rapid wicking face combined with the highly absorbent moisture locking system. The absorbent layer provides a large capacity with rapid fluid uptake. The backing is a water resistant barrier film to prevent strike-through with a high moisture vapour transfer rate prolonging wear ability.

Use

Eclipse® is placed white face down on wound surface with beige backing uppermost. For large wounds several dressings can be placed side-by-side and secured with an appropriate tape or bandage. Wear time will depend on the level of exudate but Eclipse® can be left in place for up to seven days. Eclipse® can be used under compression therapy.

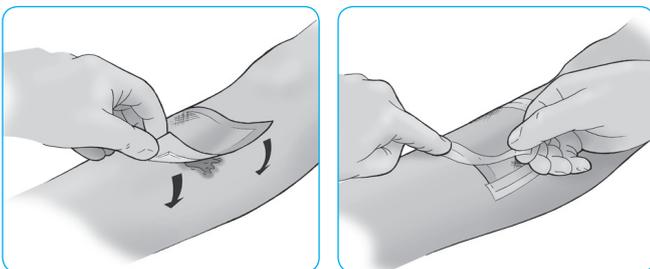

Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not cut Eclipse®.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Ordering information for Eclipse®

Size	Stock code	Pack size	NHS code	PIP code
10cm x 10cm	CR3818	20	EJE029	372-1057
15cm x 15cm	CR3769	20	EJE032	305-1349
20cm x 30cm	CR3743	20	EJE030	306-1272
60cm x 40cm	CR3808	10	EJE034	232-1131



Features

- ✓ Rapid absorption
- ✓ High capacity
- ✓ Fluid repellent backing
- ✓ Strength and durability
- ✓ Stay dry technology
- ✓ Thin and conforming

Indications

Moderate to heavily exuding wounds including:

- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Sloughy wounds
- ✓ Granulating wounds
- ✓ Post operative sutured or dehisced wounds
- ✓ Fungating wounds
- ✓ Donor site management

Care and use symbols

- Read instructions for use
- Single use only
- Do not use if packaging is broken or damaged
- Sterilised using ethylene oxide
- CE marked medical device
- Keep dry
- Store out of direct sunlight
- Do not re-sterilize

TOP TIP

Eclipse® has very similar technology to nappies, the dressing will expand with exudate, that's a great sign the dressing needs to be changed - the patient may notice it's become a little heavy.